

## NAIL FUNGUS BRUSH

### FUNGE X NAIL FUNGUS BRUSH

- Proven effective formula against nail fungus
- Kills fungus on the nail within 20 seconds!
- Scientifically proven effective in fighting/treating nail fungus
- Prevents spread of the infection and helps to prevent re-infection

**General advice:** Before using FungeX Nail Fungus Brush, read the instructions carefully, even if you have used this medical device before. The information contained in a new pack may have been revised in order to reflect the latest scientific insights and data. Nevertheless, you should use FungeX Nail Fungus Brush with care in order to obtain good results. Keep these instructions – you may need to refer to them during treatment.

**Application:** FungeX Nail Fungus Brush is a medical device that helps in the treatment of fungal nails (onychomycosis). FungeX Nail Fungus Brush has been proven to be effective against fungal nail infections. The unique formula containing an 'INSTANT KILLER', contains ingredients that penetrate the surface of the nail, killing fungi within 20 seconds. The liquid forms a layer on the nail, creating an environment that is unfavourable to fungi growth and acting as a physical barrier to prevent the spread of moulds. To achieve faster, better results, carefully filing the fungal nail once a week is recommended. If treatment is stopped, the fungal nail will return more quickly. It is therefore important to continue using FungeX Nail Fungus Brush until the fungal nail has gone.

As damp environments are one of the causes of fungal nail infections, it is strongly recommended that you wear socks made only of absorbent materials, such as cotton or wool. This is important, as moulds are commonly found in moist environments, where they can multiply. If you have a fungal nail, it is important that you keep your feet dry.

**Dosage and use (for adults only):** FungeX Nail Fungus Brush is for external use only. Apply the product to the affected area twice (mornings and evenings, for instance) until the fungal nail has fully grown out. Thanks to the 20-second instant kill formulation, the applicator brush does not become infected with the fungi, preventing infection/reinfection of the other nails and protecting against any return of the moulds. FungeX Nail Fungus Brush can be combined with the use of oral antimicrobials.

**Mode of action:** Thanks to a unique combination of ingredients, FungeX Nail Fungus Brush forms a layer, which creates an unfavourable environment for the growth of fungi in the nail plate. It also forms a physical barrier that inhibits the spread of moulds. FungeX Nail Fungus Brush reduces the nail's acidity, creating an unfavourable environment that stops the growth and spread of fungi.

**Use during pregnancy and/or breastfeeding:** No information is available regarding the use of FungeX Nail Fungus Brush during pregnancy or the breastfeeding phase. If you are pregnant or breastfeeding, consult your doctor or pharmacist before using.

**Possible side effects:** Possible side effects includes: erythema, irritation, edema, pain, nail discoloration and onycholysis; Allergic reactions.

### ⚠ WARNINGS

- For external use only.
- Fungal nail might cause erythema, irritation, edema, pain, nail discoloration and onycholysis.
- Check the list of ingredients to know if you are allergic to any of the ingredients contained in the gel.
- Avoid contact with eyes or mucous membranes.
- Do not apply to open wounds or damaged skin.
- Do not bandage tightly.
- Do not apply heat to the area of application.
- Pregnant or breastfeeding patients should seek advice of health professional before using Fungal nail.
- If you experience allergic reactions, stop using the product.
- Keep out of reach of young children.
- In case of no effect, consult a physician.

**Contains:** 1 FungeX Nail Fungus Brush with 5 ml

**Ingredients:** Aqua, Ethoxydiglycol, ISK1320®, Pentylene Glycol, Alcohol Denat., Tridiceth-9, PEG-40 Hydrogenated Castor Oil, Benzyl Alcohol, Limonene, Glycerin, Nalilact, Propylene Glycol, PVP, Xanthan Gum, Eucalyptus Globulus Leaf Oil, Lavandula Hybrida Oil, Linalool, Dimethyl Isosorbide, Citrus Aurantium Dulcis Oil, Pogostemon Cablin Oil, Disodium Phosphate, Geraniol, Citral, Biotin, Coumarin, Citric Acid, Farnesol.

All enquiries about the product should be sent to the distributor. For more information about FungeX Nail Fungus Brush, please visit [www.fungex.com](http://www.fungex.com)

FungeX is a registered trademark of TheOTCLab. FungeX Nail Fungus Brush is a class 2A medical device according to EU regulation 93/42/EC.

 Do not use after expiry date (see packaging)

Read the instructions for use before using this medical device.

Latest revision of text : September 2023

**Distributor**  
TheOTCLab  
Fred Roeskestraat 115, 1076 EE  
Amsterdam The Netherlands  
[www.theotclab.com](http://www.theotclab.com)

**Manufacturer**  
PK Benelux BV  
Vluchtoord 17, 5406 XP Uden  
The Netherlands



**CE 0481**